GEBRAUCHSANLEITUNG
USER'S MANUAL

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BECKENGURT LAP BELT

Beckengurt_DE-EN

04-21

REF Artikel /
Article

22264

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DESCRIPTION

Lap Belt

For all riders whose physical condition puts them in danger of slipping or falling out of the seat when riding or braking.

Intended use

The Lap Belt is intended for riders who require support in order to remain in an upright position in the seat. The reason can be, for example, an impaired ability to sit unsupported.

Contraindications

The Lap Belt is not suitable for any purposes that are contrary to its intended use or that could be hazardous, for example, use as a seatbelt in an automobile or use by persons not familiar with its operation. The Lap Belt must be adjusted to the user and is not suitable for riders who are not able to support the weight of their own torso.

Please observe the contraindications specified in the user manuals of the recumbent bikes and trikes.

Protection against environmental influences

The Lap Belt is made with corrosion-resistant materials. Plastics and fabric are used.

Cleaning and disinfection

The Lap Belt can be cleaned and disinfected using common household cleaning and disinfecting agents.

Applicable standards and guidelines

This is an accessory within the meaning of Regulation 2017/745 on medical devices, which as such is CE-marked.

Service life

Depending on the frequency of use, a service life of up to 8 years can be expected if the product is used in accordance with the user manual.

Reuse

Under normal conditions of use, this accessory can be reused by a new owner after inspection by an authorized dealer.

Disposal instructions

At the end of its useful life, the product should be disposed of properly, in accordance with the local waste disposal regulations (i.e. not as household waste).

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


INSTALLATION

1. Wrap the Lap Belt **(01)** around the seat frame **(02)** and secure it using the plastic buckles **(03)**.
2. The belt can be loosened **(04)** or tightened by adjusting the length with the plastic buckles.
3. The lap belt can be opened and closed quickly using the quick release fastener **(05)**.

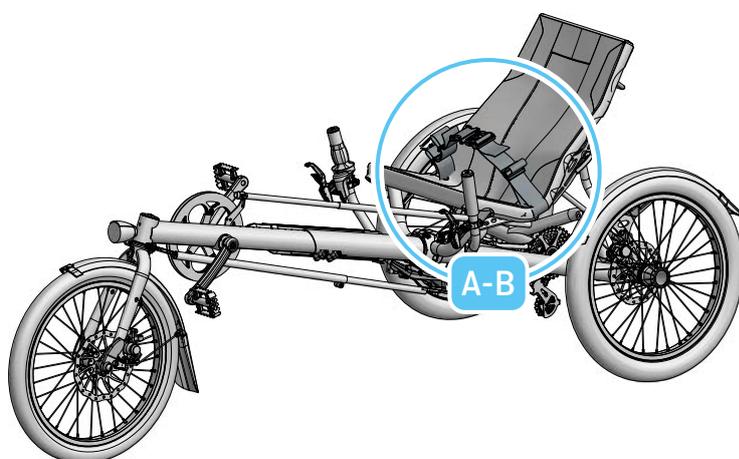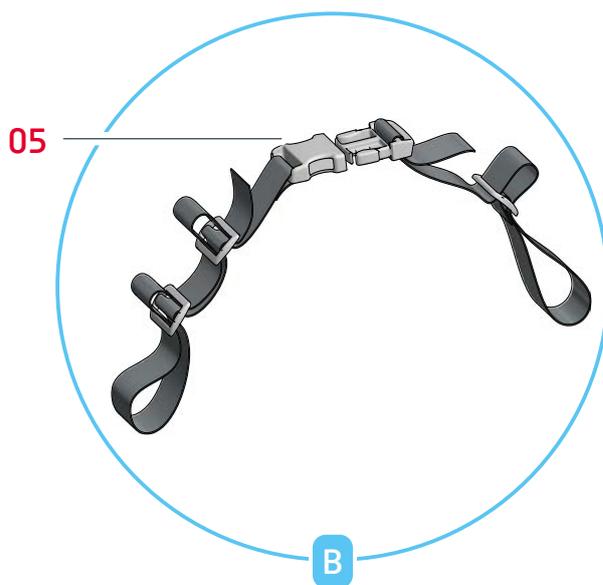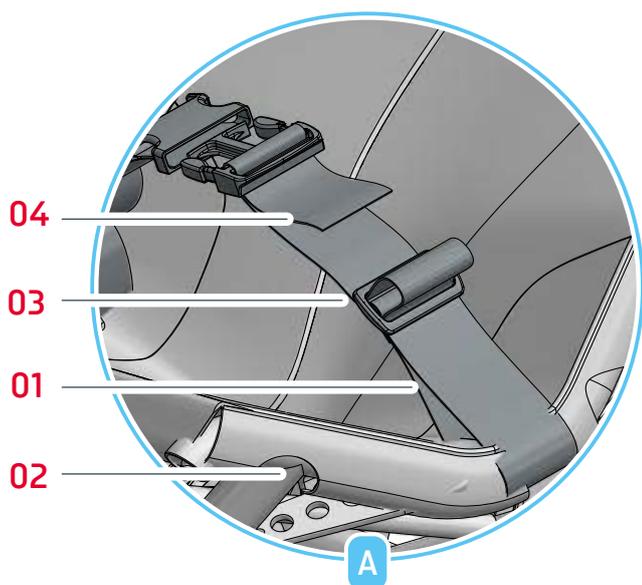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SAFETY INFORMATION

The Lap Belt is only intended for use as a means of ensuring the proper sitting position in the seat of the trike and may not be used as a seatbelt in an automobile. Defective or damaged Lap Belts may not be used and must be immediately replaced.

Information regarding incidents

Any serious incident* that occurs in connection with the product must be reported to the manufacturer and the responsible competent authorities of the member state in which the user and/or patient resides.

* A "serious incident" is any incident that had, could have had, or could have one of the following consequences, either directly or indirectly: a) the death of a patient, user, or other person, b) the serious temporary or permanent deterioration of the health condition of a patient, user, or other person, c) a serious risk to public health.

Residual risks

None known

Whom to contact with questions or problems

If you have any problems or questions about your therapeutic bike or trike, please contact your authorized dealer or HASE BIKES for assistance.